



QualiEYE is a software solution designed to replace the computer mouse, providing hands free access to your PC. The cursor on the screen is controlled by simple body movements. All you need is a standard USB Webcam. Sit in front of the computer and look at the screen. Aim the camera at any part of your body that has movements, for example your head, and move it. The USB webcam tracks the movements and the software converts them into onscreen cursor movements, providing very precise and efficient cursor control, in real time! The User is completely free and independent: no dots or headgear is required! Anyone with controlled movement of their head or fingers who requires a hands free mouse alternative can benefit. QualiEYE can be totally personalized and adjusted to any need. Just sit in front of the Web Cam and look at the screen; move your head or finger and the cursor will follow, in real time! You can control any Windows® application.

Unique advantages:

- New Innovative Headpointing Technology
- Precise hands free cursor control
- No dots or headgear is required
- Dwell selection control (Auto Click): all mouse commands available(click, double click, drag&drop,...)
- Gesture control: give commands (click, enter, page up, delete,...) with body gestures (a nod of the head to the left for a mouse click, for example)

Several features are available in order to better control the computer:

Pointing Device

The cursor move on the screen according to user's movements. A advanced settings allow the user to achieve perfect cursor control.

Mouse Click

To enable the mouse click and manage any Windows applications, the user can choose to use QualiEYE pointing device functions in combination with an external switch, the gesture mode or the Auto Click/Dwell function.

Gesture/Command Mode

User can assign 4 different commands to the 4 different head or hand gestures (right, left, up and down). He or she can choose, for example, to perform a double click by turning the head to the right side and the "Page Up" by turning the head up.

Auto-click/Dwell

QualiEYE can also be used with the Auto Click function. Simply stop the cursor for a second and the assigned command to the Auto-click (double click, right click, drag and drop,...) will be executed automatically after the Visual countdown timer runs out.



Limit the area of capture

Limit the area of capture (for example the eye) and only the movements inside the limited area are considered by QualiEYE to avoid any interferences from background movements. This feature is the perfect solution for users that want to perform the click with only the blink of an eye, or a small movement of a cheek, lips, eyebrow, finger, foot, etc.

Advanced settings

Settings can be fully personalized in order to increase user's ability in controlling the cursor. Speed, sensitivity, smoothness and non-linearity adjustments are available.

Multi-Language

The software, the On-line Help and the User Manual are available in several languages including English, French, German, Italian and many others to come.

Multi-User

QualiEYE is Multi-User software. When you run the program it will load the setting of the same user logged in into Windows.



Access Technology and Plug & Play Computer Solutions for People with Disabilities

Toll Free: 800-USA-EVAS
Toll Free: 800-872-3827
Voice: 401-596-3155
FAX: 401-596-3979



Email: Contact@evas.com
techhelp@evas.com
TTY: 401-596-3500
www.evas.com